

QUICK REFERENCE FOR PEDIATRIC BLADDER VOLUME

BladderScan®

The Pediatric BladderScan is optimized for children up to 48" (122cm) tall and weighing up to 60lbs (27kg).

Note: To turn on the scanning unit when it is in sleep mode, press and release any button and wait for the chirp.

1

Select Gender


Select the female option **ONLY** for female patients

Select gender




2

Apply Sontac® Gel Pad

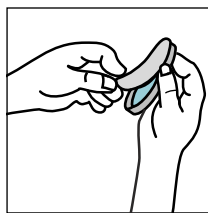

Open gel pad package

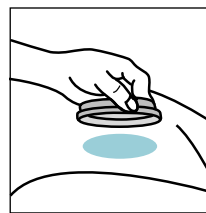

Place gel pad on the abdomen of the patient, one inch (3 cm) above the symphysis pubis




Place the scanhead on the gel pad

3

Aim Toward Bladder

Aim the scanhead slightly downward toward the patient's coccyx (tail bone)

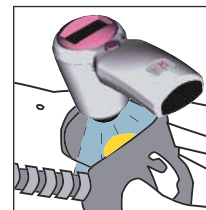

Aim toward bladder

4

Press Scan Button

Scan button



5

Verify Aim


5A

Flashing Arrows = Re-Aim




Flashing arrow appears

Example:

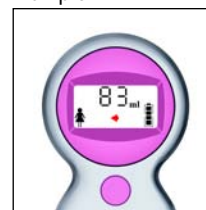

1) Arrow points left

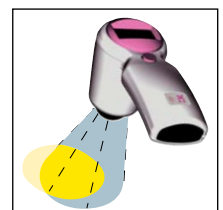

2) Aim left, then scan again

5B

Solid Arrows or No Arrows = Accept Scan Re-Aiming Not Necessary



Solid arrow or no arrow



BVI 6200

DIAGNOSTIC
ULTRASOUND

www.dxu.com <https://my.scanpoint.com>

For more information, call 800.331.2313 (in the U.S. and Canada) or contact your local Diagnostic Ultrasound representative.

BladderScan® and Sontac® are registered trademarks of Diagnostic Ultrasound Corporation. All rights reserved.

0900-0681-03-60

